




RUNRITE ELECTRONICS


Operation Explanation for R-3000 Alcohol Tester

Scale panel explanation:

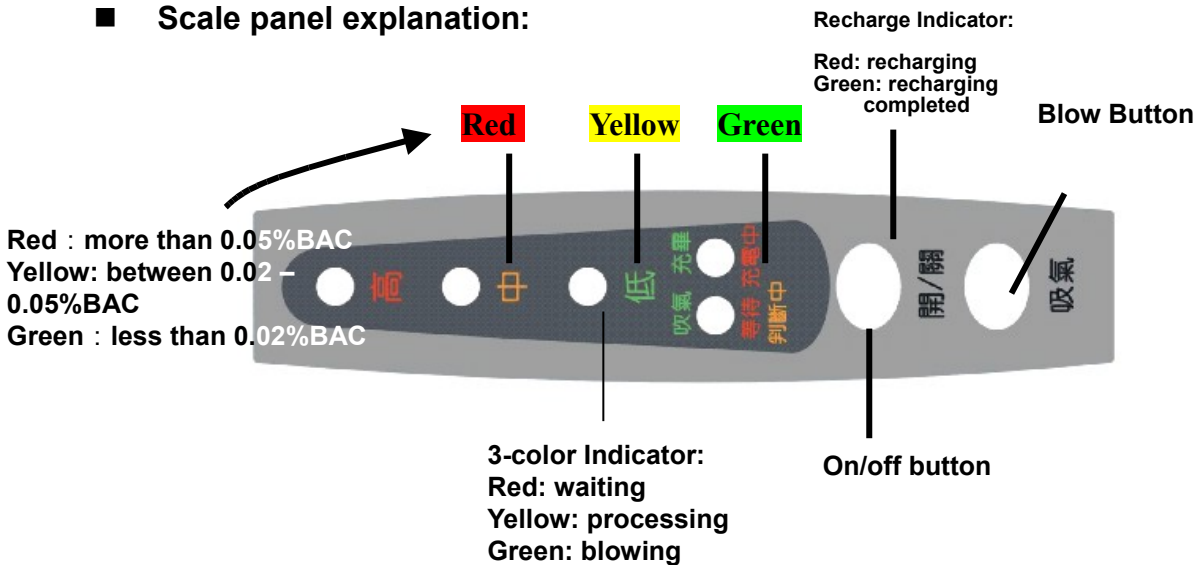

Operating manual :

Within 3 to 5 cm. from the mouthpiece, press Test button then blow continuously and strongly for 2 seconds, then wait for the test result. While blowing, press Test button continuously.

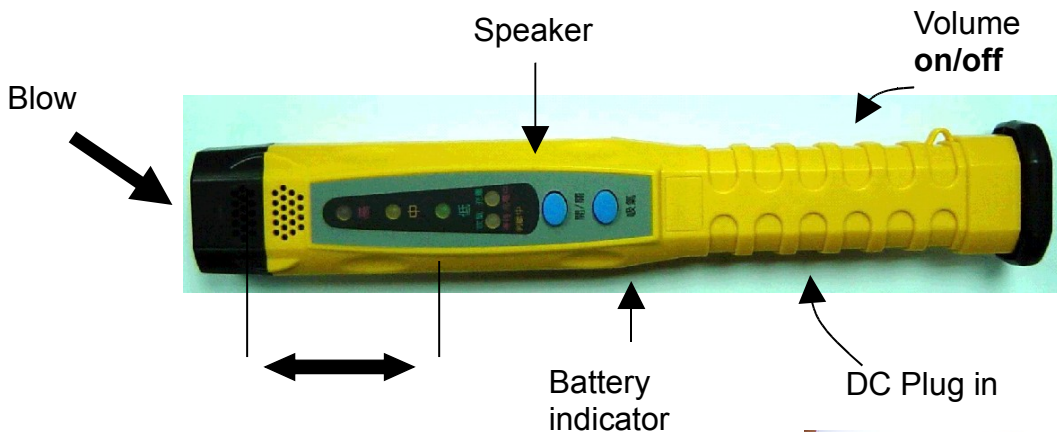

- Results of alcohol concentration tests, provided by this device, are for reference only.
- One complete set includes one Alcohol Tester, one AC Battery Charger Adapter and one Cigarette Plug Adapter.



R-3000 AlcoLight

Features:

- Fuel cell sensor reacts high sensitivity to alcohol
- Fast and easy testing operation – quick detecting and recovery in a short time.
- Two modes of operation – active by blowing and passive by sucking.
- Testing results announced by voice.
- No mouthpieces needed.
- No regular calibration adjustments needed (annual calibration)
- Low power needed. Saving mode activates in 3 minutes automatically.
- Battery charging system is included.

Status Light:

The “Wait” and “Ready” light indicates the below status:

- Red** Wait
- Green** Ready for breath

The “Batt Charging” and “Batt Full” indicates the below status:

- Red** Battery is under charge
- Green** Battery is fully charged

The “Battery Load” lights indicate the status of battery loading:

- Red** 20% or lower
- Amber** 40%
- Green** 100% to 60%

Alcohol Light:

There are three lights to indicate the alcohol concentration after a test is completed:

- Green** with 1 buzz No alcohol (< 0.5 0/00)
- Amber** with 3 buzzes Low concentration (0.5 ~ 0.8 0/00)
- Red** with 5 buzzes High concentration (>0.8 0/00)

The **AlcoLight** can be used in many testing situations, including:

- Drivers on road
- Guests in bars
- Employee testing

